

	文件类型	技术文件	文件版本	V3.4.0
	项目代号	/	页 码	第0页 共 2页
	文件编号	GL DM-56-17-en		

文件名称:

CONTROL DM-RET RET0723 2023-09-10

适用范围: 1116 项目、7003 项目

编制: 王雨婷 日期: 2023.07.11	审核: 郑剑通 日期: 2023.07.11	批准: 郑剑通 日期: 2023.07.11
---------------------------	---------------------------	---------------------------

修 订 记 录

版本	TCN/ECR/PCN 编号	修订内容概要	修改人	批准人	批准日期
V3.4.0	/	新建	王雨婷	郑剑通	2023.07.11

发至: 总经办 质量部 财务部 营销部 客服 计划 采购 工程 仓库
设备研发 试剂研发 物料质量 设备生产 试剂生产 试剂质检 设备质检

由试剂研发

DM-RET

HEMATOLOGY CONTROL

CONTROL


📅 2023-06-13

🕒 2023-09-10


Applicable Instruments	Parameter	Unit	LOT RET0723L	LOT RET0723N	LOT RET0723H
DYMIND DH-600 DH-602 DH-605 DH-610 DH-612 DH-615 (Technical File Version A2.0 or higher)	RET#	$\times 10^{12}/L$	0.0409 ± 0.0205	0.2395 ± 0.1437	0.2928 ± 0.1464
	RET%	%	0.72 ± 0.36	5.07 ± 2.03	12.36 ± 4.94
	LFR	%	90.0 ± 27.0	89.7 ± 26.9	89.7 ± 26.9
	MFR	%	8.0 ± 12.0	8.4 ± 12.6	8.4 ± 12.6
	HFR	%	2.1 ± 3.2	2.0 ± 3.0	1.9 ± 2.9
	IRF	%	10.0 ± 15.0	10.3 ± 15.5	10.3 ± 15.5
	RHE	pg	16.7 ± 6.7	15.7 ± 6.3	14.6 ± 5.8

【NOTE】


1. The controls should be stored in refrigerator (2°C~8°C). After opening, it will keep stable for 14 days when it is stored airtight at 2°C~8°C.
2. Before mixing and running the control after take it out from the refrigerator, please keep it at least 15 minutes until reaching room temperature(15°C~30°C).
3. Controls must be well mixed before using. Please mix gently, to avoid cells rupture and/or generating bubbles.
4. After using, put the controls back into the refrigerator to prevent contamination and evaporation.

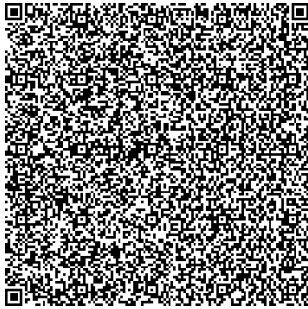

RET0723L

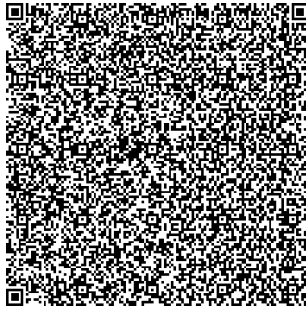

RET0723N

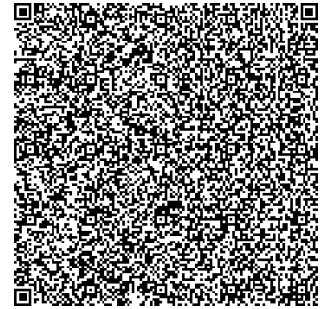

RET0723H

DM-RET

HEMATOLOGY CONTROL

CONTROL

📅 2023-06-13

🕒 2023-09-10

Applicable Instruments	Parameter	Unit	LOT RET0723L	LOT RET0723N	LOT RET0723H
DYMIND DM60 Vet DM61 Vet DM62 Vet DM63 Vet (Technical File Version A1.1 or higher)	RET#	×10 ¹² /L	0.0210 ±0.0168	0.1383 ±0.1107	0.1634 ±0.1308
	RET%	%	0.37 ±0.30	2.93 ±2.34	6.74 ±5.39

【NOTE】

1. The controls should be stored in refrigerator (2°C~8°C). After opening, it will keep stable for 14 days when it is stored airtight at 2°C~8°C.
2. Before mixing and running the control after take it out from the refrigerator, please keep it at least 15 minutes until reaching room temperature(15°C~30°C).
3. Controls must be well mixed before using. Please mix gently, to avoid cells rupture and/or generating bubbles.
4. After using, put the controls back into the refrigerator to prevent contamination and evaporation.



RET0723L



RET0723N



RET0723H