

	文件类型	技术文件	文件版本	V2.2.0	
	项目代号	/	页 码	第 0 页 共 1 页	
	文件编号	GL DM-56-62-en			
文件名称： <p style="text-align: center;"><b>CONTROL DM-BF BF0324 2024-05-10</b></p> 适用范围：1117 项目					
编制：刘嘉 日期：2024.03.11		审核：刘金盈 日期：2024.03.11		批准：郑剑通 日期：2024.03.11	
修 订 记 录					
版本	TCN/ECR/PCN 编号	修订内容概要	修改人	批准人	批准日期
V2.2.0	/	新建	刘嘉	郑剑通	2024.03.11
发至：	<input type="checkbox"/> 总经办 <input type="checkbox"/> 质量部 <input type="checkbox"/> 财务部 <input type="checkbox"/> 营销部 <input type="checkbox"/> 客服 <input type="checkbox"/> 计划 <input type="checkbox"/> 采购 <input type="checkbox"/> 工程 <input type="checkbox"/> 仓库 <input type="checkbox"/> 设备研发 <input type="checkbox"/> 试剂研发 <input type="checkbox"/> 物料质量 <input type="checkbox"/> 设备生产 <input type="checkbox"/> 试剂生产 <input type="checkbox"/> 试剂质检 <input type="checkbox"/> 设备质检				
由试剂研发					

# DM-BF

## HEMATOLOGY CONTROL

**CONTROL**

📅 2024-02-11

🕒 2024-05-10

Applicable Instruments	Parameter	Unit	<span style="border: 1px solid black; padding: 2px;">LOT</span> BF0324L	<span style="border: 1px solid black; padding: 2px;">LOT</span> BF0324N	<span style="border: 1px solid black; padding: 2px;">LOT</span> BF0324H
DYMIND DH-800 series DH-800 CRP series	WBC-BF	10 <sup>9</sup> /L	0.093 ±0.050	0.239 ±0.150	0.897 ±0.200
DH-800 CS series (Technical File Version A1.1 or higher)	RBC-BF	10 <sup>12</sup> /L	0.023 ±0.020	0.067 ±0.060	0.450 ±0.100

**【NOTE】**

1. The controls should be stored in refrigerator (2°C~8°C). After opening, it will keep stable for 30 days when it is stored airtight at 2°C~8°C.
2. Before mixing and running the control after take it out from the refrigerator, please keep it at least 15 minutes until reaching room temperature(15°C~30°C).
3. Controls must be well mixed before using. Please mix gently, to avoid cells rupture and/or generating bubbles.
4. After using, put the controls back into the refrigerator to prevent contamination and evaporation.



BF0324L



BF0324N



BF0324H