

	文件类型	技术文件	文件版本	V2.2.0
	项目代号	/	页 码	第0页 共 1页
	文件编号	GL DM-56-17-en		

文件名称:

**CONTROL DM-RET RET0424 2024-06-10**

适用范围: 1117 项目

编制: 刘嘉 日期: 2024.04.07	审核: 刘金盈 日期: 2024.04.07	批准: 郑剑通 日期: 2024.04.07
--------------------------	---------------------------	---------------------------

修 订 记 录

版本	TCN/ECR/PCN 编号	修订内容概要	修改人	批准人	批准日期
V2.2.0	/	新建	刘嘉	郑剑通	2024.04.07

发至: 总经办 质量部 财务部 营销部 客服 计划 采购 工程 仓库  
设备研发 试剂研发 物料质量 设备生产 试剂生产 试剂质检 设备质检

由试剂研发

# DM-RET

## HEMATOLOGY CONTROL

**CONTROL**

2024-03-13

2024-06-10

Applicable Instruments	Parameter	Unit	LOT RET0424L	LOT RET0424N	LOT RET0424H
DYMIND	RET#	$\times 10^{12}/L$	0.0392 $\pm 0.0196$	0.1289 $\pm 0.0774$	0.2399 $\pm 0.1200$
DH-800 series	RET%	%	1.15 $\pm 0.58$	3.78 $\pm 1.51$	6.99 $\pm 2.79$
DH-800 CRP series	LFR	%	89.6 $\pm 27.0$	89.8 $\pm 27.0$	89.5 $\pm 27.0$
DH-800 CS series (Technical File Version A1.3 or higher)	MFR	%	7.4 $\pm 11.0$	7.2 $\pm 11.0$	7.5 $\pm 11.0$
	HFR	%	3.0 $\pm 5.0$	3.0 $\pm 5.0$	3.0 $\pm 5.0$
	IRF	%	10.4 $\pm 16.0$	10.2 $\pm 16.0$	10.5 $\pm 16.0$
	RHE	pg	28.4 $\pm 10.0$	27.6 $\pm 10.0$	27.9 $\pm 10.0$

**【NOTE】**

1. The controls should be stored in refrigerator (2°C~8°C). After opening, it will keep stable for 14 days when it is stored airtight at 2°C~8°C.
2. Before mixing and running the control after take it out from the refrigerator, please keep it at least 15 minutes until reaching room temperature(15°C~30°C).
3. Controls must be well mixed before using. Please mix gently, to avoid cells rupture and/or generating bubbles.
4. After using, put the controls back into the refrigerator to prevent contamination and evaporation.



RET0424L



RET0424N



RET0424H