





# DM-BF

## HEMATOLOGY CONTROL

**Reference Values provided by DYMIND**
**CONTROL**
 2025-04-12

 2025-07-10

Applicable Instruments	Parameter	Unit	LOT BF0525L	LOT BF0525N	LOT BF0525H
DYMIND DH-800 series DH-800 CRP series DH-800 CS series (Technical File Version A1.3 or higher)	WBC-BF	10 <sup>9</sup> /L	0.084 ±0.050	0.260 ±0.150	1.070 ±0.200
	RBC-BF	10 <sup>12</sup> /L	0.026 ±0.020	0.068 ±0.060	0.470 ±0.100

**【NOTE】**

1. The controls should be stored in refrigerator (2°C~8°C). After opening, it will keep stable for 30 days when it is stored airtight at 2°C~8°C.
2. Before mixing and running the control after take it out from the refrigerator, please keep it at least 15 minutes until reaching room temperature(15°C~30°C).
- 3.Controls must be well mixed before using. Please mix gently, to avoid cells rupture and/or generating bubbles.
4. After using, put the controls back into the refrigerator to prevent contamination and evaporation.



BF0525L



BF0525N



BF0525H